# Natural Dermal Template

## **EN** Product description

**Endoform® Natural Dermal Template** is an intact extracellular matrix (ECM). Derived from ovine (sheep) forestomach tissue, this advanced wound care device is non-reconstituted collagen, thus it retains the innate biological structure of the native ECM associated macromolecules including elastin, fibronectin, glycosaminoglycans and laminin. When rehydrated with wound exudate or sterile saline, **Endoform® Natural Dermal Template** transforms into a soft conforming sheet, which is naturally incorporated into the wound over time.

Not made with natural rubber latex.

### Intended use

**Endoform® Natural Dermal Template** is a sterile, single use ovine forestomach-derived extracellular matrix intended to cover, protect, and provide a moist wound environment.

### Indications for use

**Endoform® Natural Dermal Template** is indicated for the management of wounds including:

- Partial and full thickness wounds
- Pressure ulcers
- Venous ulcers
- Diabetic ulcers
- Chronic vascular ulcers
- Tunneled/undermined wounds
- Surgical wounds (donor sites, grafts, post Moh's surgery, post laser surgery, podiatric, wound dehiscence)
- Traumatic wounds (abrasions, lacerations, first and second-degree burns, and skin tears)
- Draining wounds

### Contraindications

- **Endoform® Natural Dermal Template** is derived from an ovine (sheep) source and should not be used on patients with known sensitivity to ovine (sheep) derived material.
- **Endoform® Natural Dermal Template** is not indicated for use on third degree burns.

### Precautions

- Do not apply to wounds with uncontrolled clinical infection, acute inflammation, excessive exudate, or bleeding
- **Endoform® Natural Dermal Template** is supplied sterile. Do not use if the pouch seal is broken.
- Discard the device if mishandling has caused possible damage or contamination.
- Single use product. Do not attempt to re-sterilize. Discard all unused portions. Device reuse may contribute to wound infection.
- Always handle **Endoform® Natural Dermal Template** using aseptic technique.

### Storage

**Endoform® Natural Dermal Template** should be stored between 15°C/59°F–40°C/104°F in a clean and dry area.

### Directions for use

These recommendations are designed to serve only as a general guideline. They are not intended to supersede institutional protocols or professional clinical judgment concerning patient care.

#### 1) Wound Preparation

- Prepare the wound bed by cleansing, irrigation and if necessary debridement to ensure the wound is free of debris, necrotic tissue or infected tissue.

#### 2) Device Application

- Select a sheet of the dermal template which is slightly larger than the wound and apply aseptically.
- Endoform® Natural Dermal Template** can be applied as a whole sheet or trimmed so that it contacts the wound margins. Multiple sheets can be used to cover the entire wound bed.
- For ease of handling, apply the dermal template by placing the dry material in the wound and rehydrating with exudate or sterile saline. When hydrated **Endoform® Natural Dermal Template** transforms into a soft conforming sheet. Ensure that the dermal template conforms to the underlying wound bed.
- To protect **Endoform® Natural Dermal Template** from adhering to the secondary dressing, consider applying a non-adherent dressing over the dermal template to help protect the tissue while facilitating an optimal moist wound healing environment.
- Secure the dermal template using an appropriate secondary dressing. The overlying dressing should be changed according to standard of care taking into account the level of exudate.
- The biodegradable **Endoform® Natural Dermal Template** is naturally absorbed and incorporated into the wound over time. **Endoform® Natural Dermal Template** may last up to 7 days in the wound, depending on the wound environment.
- Endoform® Natural Dermal Template** can be used in conjunction with compression therapy and negative pressure wound therapy under the supervision of a health care provider.

#### 3) Device Changes

- Duration of treatment and reapplication is determined by the physician and depends upon the wound type and conditions.
- Reapply when the dermal template has been incorporated into the wound. It is recommended to assess the wound within 72 hours. **Endoform® Natural Dermal Template** may last up to 7 days in the wound, depending on the wound environment.
- Carefully cleanse the wound surface in accordance with established procedures. Do not attempt to remove the off-white to golden gel that forms from residual **Endoform® Natural Dermal Template** in the wound as it contains extracellular matrix components that assist in wound healing.
- If areas of the dermal template are dry upon inspection, rehydrate with sterile saline and leave in place.
- It is not necessary to remove any residual **Endoform® Natural Dermal Template** during dressing changes. However, if the product has been overlapped onto the peri-wound area, the remaining loose product that has not incorporated into the wound may be gently removed around the edges if desired.
- Change the secondary dressings as needed, and when **Endoform® Natural Dermal Template** is re-applied.

Refer to Eudamed for a summary of safety and clinical performance

Report any serious incident related to **Endoform® Natural Dermal Template** to the Aroa Biosurgery via Healthlink Europe as well the applicable Member State competent authority.

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©2019 Aroa Biosurgery Ltd
**Endoform®** is a registered trademark of **Aroa Biosurgery Ltd.**
Aroa Biosurgery Ltd.